

Usage Instructions for Soft Contact Lenses

IMPORTANT INFORMATION

Please read the following information carefully and keep it for reference.

INTRODUCTION

These lenses are intended for daily wear, single patient use, or as recommended by your eye care practitioner. Each STERILE lens is supplied in a sealed container and no special storage conditions are required before opening. However, normal room temperature storage is recommended to maintain the overall condition of the lenses.

INTENDED USE

The soft contact lens is for daily wear use for the correction of visual acuity in aphakic or non-aphakic persons with non diseased eyes that are myopic or hyperopic and may exhibit refractive astigmatism that does not interfere with visual acuity.

CONTRAINDICATIONS (Reasons Not To Use)

DO NOT USE the soft contact lenses when any of the following conditions exist :

- Acute and sub-acute inflammation or infection of the anterior chamber of the eye
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids
- Severe insufficiency of lacrimal secretion
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses
- Allergic reactions of ocular surfaces
- Any active corneal infection (bacterial, fungal, protozoal or viral)
- If eyes become red or irritated

CAUTION

To be dispensed and sold by a qualified and licensed eyecare practitioner only.

WARNINGS

Do

- Use recommended soft contact lens care products and follow the manufacturers' instructions for use
- Remove lenses and seek advice if any of the following occur : pain, red eyes, blur vision, excessive tears and sensitivity to light
- Visit your eye care practitioner regularly for check-ups

Do Not

- Use any unusual fluids with your lenses or lens case e.g. tap water, mineral water and solutions formulated for lenses other than soft contact lenses
- Neglect hygiene for lenses and storage cases. (Poor hygiene during use and care of lenses introduced the risk of eye infections.)
- Swim with contact lenses unless watertight goggles are worn.
- Use any new eye care products without consulting your eye care practitioner or doctor

PRECAUTIONS

- Some medical conditions may affect the ability to wear lenses successfully
- Lenses can be associated with clinical conditions e.g. corneal anoxia, allergic response, trauma and bacterial infection.
- Keep lenses and lens care products out of reach of children
- Eye Care Professionals should instruct the patient to remove lenses immediately if the eyes become red or irritated.

IN AN EMERGENCY SITUATION OR IF YOU ARE IN DOUBT WITH THE INFORMATION ON THIS LEAFLET, PLEASE CONSULT YOUR EYE CARE PRACTITIONER OR EYE DOCTOR FOR ADDITIONAL ADVICE.

PREPARATION

- Always wash and dry your hands before handling lenses so as not to contaminate them.
- Check packaging of new containers for signs of damage. Do not use if the original packaging is damaged or broken. Please return it immediately to your dispensing eyecare practitioner.

INSERTION

- Remove lens from case with finger and place on fingertip and ensure lens is not inverted (inside out).
- Pull down lower lid, lean forward, look into mirror with both eyes and then draw up upper lid and look downwards.
- Press lens lightly onto the eye and gently release the lids.
- Repeat same procedure with the other lens.

IF ABNORMAL DISCOMFORT IS FELT – remove lens and check it is not inverted.

IF DISCOMFORT PERSISTS – remove lens immediately and seek advice from your eye care practitioner.

REMOVAL

- Pull down lower lid; slide lens down to white of eye while looking up, gently pinch lens and remove.
- Repeat same procedure with the other lens

CLEANING AND DISINFECTING

- Always clean, rinse, disinfect and store lenses using products recommended by your eyecare practitioner. Follow the instructions for use on the bottles. Lenses must be stored fully submerged in a clean lens case using fresh solutions everyday.
- If lenses are only worn occasionally, seek storage advice from your eye care practitioner.
- Contact lenses should be discarded after the recommended wearing period prescribed by your eyecare practitioner.

In case of complaint

Please contact your eye practitioner with details of your concern. Give details of batch references to assist with prompt processing of concerns. Your statutory rights are not affected.

CONTACT LENS PATIENTS SHOULD FOLLOW INSTRUCTIONS GIVEN BY THEIR EYE CARE PRACTITIONER STRICTLY AND DISCARD THEIR CONTACT LENSES AFTER THE WEARING PERIOD (MODALITY) PRESCRIBED BY THE EYE CARE PRACTITIONER.



IGEL Far East Pte Ltd
135 Joo Seng Road, #05-01
PM Industrial Building
Singapore 368363

Tel: 65-6282 7055
Fax: 65-6382 5077
Email: ife@igel.com.sg

IGEL (Malaysia) Sdn Bhd
No. 25, Jalan 6/33 B, Batu 61/2
Jalan Kepong, 52000 Kuala
Lumpur, Malaysia

Tel: 603-6253 3888
Fax: 603-6258 8477
Email: igelmalaysia@igel.com.sg

**IGEL Optical Science &
Technology (Shanghai) Co. Ltd**
3F No.6 Lane, Tong Pu Road
#1130 Putuo Chang Zheng
Industrial Zone, Shanghai 20333,
China

Tel: 86-21-5270 0383
Fax: 86-21-6391 6056
Email: igelsh@igel.com.sg